



Vitique

Veneer Cementation

Expert Veneer Cementation

Vitique is a premium resin cement developed especially to enhance the production of perfect veneers. This high quality cement offers several innovations unmatched by other veneer cements such as superior color stability for long lasting beautiful veneers; consistent ability to reflect the shade of the veneer and facilitate the natural transparency of the tooth; optimum flow characteristics allowing for uncomplicated placement of the thinnest veneer. Vitique's lack of sensitivity to ambient light allows for relaxed, stress-free placement of multiple veneers.

FEATURES & BENEFITS

- Outstanding shade reproduction and shade selection
- Exact shade match between Try-In and cement
- Remove excess material with ease
- High color stability
- Practical veneer tip
- Complete system for veneer cementation

DMG America has perfected an exact color match between Vitique's Try-In Pastes and the corresponding cements thus ensuring a precise preview of the final case.




The Vitique Cementation Kit

The Vitique Cementation Kit contains cements, matching Try-In pastes and veneer tips.



The Vitique Accessory Kit

The Vitique Accessory Kit contains everything you need for perfect veneer cementation: etching gel, silane and bonding system.

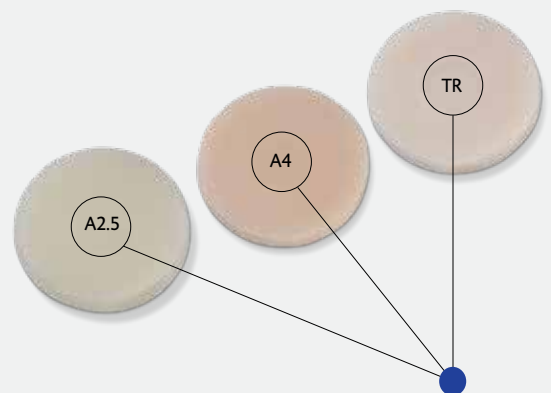

Perfect color coordination.

The center of each color circle has been filled with Vitique Try-In paste.

Remove Excess Material with Ease

Vitique's advanced formula allows for effortless clean up. The perfectly adjusted gel-phase (after tacking) provides an easy, fast and clean removal of excess cement.



Practical Veneer Tip

A special veneer tip has been developed to assist in delivering the ideal amount of cement onto the veneer. It can be used to evenly apply a flat strip of cement, both conveniently and quickly.



The TECO Total-Etch bonding system

The bonding system contained in the Accessory Kit is DMG's TECO Total-Etch; a single bottle system that provides excellent bond strength and helps to ensure reliable veneer cementation.

Achieving results with Vitique



Initial situation




Simple removal of excess Vitique



Final result with ceramic veneers

Photos courtesy of: Dr. Paolantoni, Italy

The shear bond strength (SBS) of various adhesive systems on etched enamel

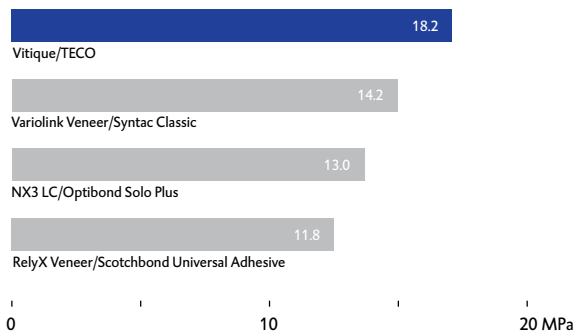

Source: Internal measurements, DMG, Hamburg 2013.

Variolink Veneer/Syntac Classic are registered trademarks of Ivoclar Vivadent, INC. NX3 LC/Optibond Solo Plus are registered trademarks of Kerr Corporation. Rely X Veneer/Scotchbond Universal Adhesive are registered trademarks of 3M Espe.

Vitique Silane

Silane is a two component system which consists of an adhesive silane and an activator. The components are mixed in a 1:1 ratio directly before application, ensuring a fresh mix, resulting in a better bond.





Ordering Information

Item#	Product	Content
213080	Vitique Veneer System	3 Syringes @ 6g Esthetic Resin Cement - Shades: A2.5, B1, TR 3 Syringes @ 3.9g Try-In Paste - Shades: A2.5, B1, TR 1 Syringe @ 2ml Etching Gel 1 Bottle @ 3ml Vitique Silane Activator 50 Veneer Tips 1 Bottle @ 5ml TECO 1 Bottle @ 3ml Vitique Silane Adhesive Accessories
213082	Vitique Cementation Kit	3 Syringes @ 6g Esthetic Resin Cement - Shades: A2.5, B1, TR 3 Syringes @ 3.9g Try-In Paste - Shades: A2.5, B1, TR 50 Veneer Tips
213082	Vitique Accessory Kit	1 Syringe @ 2ml Etching Gel 1 Bottle @ 3ml Vitique Silane Activator 1 Bottle @ 3ml Vitique Silane Adhesive 1 Bottle @ 5ml TECO Accessories
213038	Refill Cement - Shade A1	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade A1)
213039	Refill Cement - Shade A2.5	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade A2.5)
213040	Refill Cement - Shade A4	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade A4)
213041	Refill Cement - Shade B1	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade B1)
213042	Refill Cement - Shade Bleach Light	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade Bleach Light)
213043	Refill Cement - Shade Transparent	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade Transparent)
213044	Refill Cement - Shade White	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade White)
213045	Refill Cement - Shade Pink	1 Syringes @ 6g Esthetic Resin Cement 10 Veneer-Tips (Shade Pink)
213048	Refill Try-In Paste - Shade A1	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade A1)
213049	Refill Try-In Paste - Shade A2.5	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade A2.5)
213050	Refill Try-In Paste - Shade A4	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade A4)
213051	Refill Try-In Paste - Shade B1	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade B1)
213052	Refill Try-In Paste - Shade Bleach Light	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade Bleach Light)
213053	Refill Try-In Paste - Shade Transparent	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade Transparent)
213054	Refill Try-In Paste - Shade White	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade White)
213055	Refill Try-In Paste - Shade Pink	1 Syringes @ 3.9g Try-In Paste 10 Veneer-Tips (Shade Pink)
213057	Veneer Ribbon Tip	100 pack
213118	Vitique Silane	1 Bottle @ 3ml Adhesive, 1 Bottle @ 3ml Activator, Accessories



DMG America, LLC
65 Challenger Road, Suite 340, Ridgefield Park, NJ 07660
tel: 800.662.6383 email: info@dmg-america.com
web: www.dmg-america.com

