Honigum-Light

Inhalt | Contents

DE – Gebrauchsinformation	4
EN – Instructions for use	11
FR – Mode d'emploi	17
IT – Istruzioni per l'uso	24
ES – Instrucciones de uso	31
PT – Instruções de utilização	38
NL – Gebruiksaanformatie	45
DA – Brugsanvisning	52
SV – Bruksanvisning	58
PL – Instrukcja użycia	65
RU – Информация по применению	72
TR – Kullanım bilgileri	80

Produktbeschreibung

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Vinylpolysiloxane, Siliciumdioxid, Hydrogenpolysiloxane, Pigmente, Additive, Platinkatalysator

Klassifikation

ISO 4823:2000, Type 3: Light-bodied consistency

Handelsformen

4 Kartuschen à 25 ml Paste 16 Automix-Tips, 16 Intraoral-Tips	
	REF 909539
 Fast	REF 909639

2 Kartuschen à 50 ml 12 Automix-Tips, 12 Intraoral-Tips	
	REF 909831
 Fast	REF 909832

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16 Kartuschen à 25 ml Paste 50 Automix-Tips, 50 Intraoral-Tips	
	REF 909690
 Fast	REF 909691

Product description

Honigum-Light is a correction impression material based on A-silicones.

The material is available in the Automix safety cartridge.

Indications

- Crown and bridge impressions
- Inlay and onlay impressions

Basic Safety Instructions

- For dental use only!
- Keep out of reach of children!

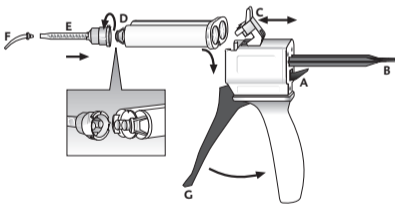
Interactions with other materials

- Rinse residues of retraction solutions thoroughly with water, otherwise these in some instances can increase the setting time.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swell-

ing of the material and may therefore lead to inaccuracies in the cast.

- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE based gloves.
- Do not mix Honigum-Light with condensation curing silicone or polyether impression materials.

Using the Automix cartridge

Insert cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the cartridge line up with those on the dispensing gun.

Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are available as refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

4. Attach an Intraoral-Tip [F], if applicable.

Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [G] of the dispensing gun and it is then applied directly.

Note: After the application, leave the used mixing tip on the cartridge to cap unused material!

Remove cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Instructions for use

- ▶ For the individual times applicable to the materials used, refer to the “Technical data” sec-

tion. Follow the associated instructions for use!

- In order to improve the adhesion of the pre-impression material to metal or plastic trays, a tray adhesive for A-silicones should be applied (e.g. Tray Adhesive, DMG).
- The Intraoral-Tips are to be used for direct, intrasulcular application around the preparation.

Recommended Use

Double-mix impressions

Fill the tray with Honigum-Putty, -Heavy or -Mono. When doing so, hold the impression tray at a slight angle beneath the mixing tip. Follow the associated instructions for use!

Note: To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

1. Syringe the preparation with Honigum-Light.

Note: The filling of the tray and syringing should be completed at the same time, because the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

2. Position the tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
3. Remove the impression from the mouth once the setting time has ended; see “Removal” on page 14

Correction impression

Note: No more than 30 minutes are permitted to elapse between the pre-impression and correction impression nor should alcohol be used for cleaning during this time, otherwise it is not possible to guarantee a firm bond between the two impression materials.

1. Cut generous drainage channels into the pre-impression for the wash and remove imperfections.

2. After repositioning, thoroughly rinse the impression with water and then dry.
3. Apply Honigum-Light around the preparation and on the pre-impression.
4. Position the tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
5. Remove the impression from the mouth once the setting time has ended; see “Removal” on page 14

Removal

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.
- ▶ Then proceed as described under “Cleaning and disinfection of the final impression” on page 14.

Cleaning and disinfection of the final impression

- ▶ Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.

- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression materials.
- ▶ Follow the instructions of the disinfectant solution's manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting; see “Storage of the impression” on page 15
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873; Type 4, e.g. Fujirock II, GC).

The impression can be copper-plated or silver-plated.

Storage of the impression

- ▶ Store at room temperature (15–25 °C/59–77 °F) in a dry place!
- ▶ Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

Technical data

Honigum-Light		 Fast
Working time ¹ [min]	≤ 2:15	≤ 1:30
Intraoral setting time ² [min]	≥ 3:30	≥ 2:00
Dimensional change after 24 h [%]	≤ 0.5	≤ 0.5
Compression set [%]	≤ 0.35	≤ 0.35
Shore-A hardness	≈ 47	≈ 47

Honigum-Light		 Fast
Strain in compression [%]	≈ 4.5	≈ 3.9

¹The time stated applies at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

²The times indicated apply from the start of mixing for a temperature of 35 °C.

Disinfection of the Automix-Dispenser

The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser is also autoclavable.

Storage and shelf life

- ▶ Store at room temperature (15–25 °C/59–77 °F) in a dry place!
- ▶ Do not store in the fridge or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ After the application, leave the used mixing tip on the cartridge to cap unused material!
- ▶ Do not use after the expiration date!

Composition

Vinyl polysiloxanes, silicium dioxide, hydrogen polysiloxanes, pigments, additives, platinum catalyst

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DMG Chemisch-Pharmazeutische Fabrik GmbH