



Kolorz Neutral Fluoride Foam

Neutral Sodium Fluoride

**Contains 2.0% Sodium Fluoride
(0.9% Fluoride Ion)**

Available in: Triple Mint

Indications:

For the topical application of fluoride in a neutral base to aid in the protection against dental caries. Ideal for patients with extensive esthetic restorations and/or intolerance to acidic fluorides, Kolorz Neutral Foam requires a 4-minute application.

Shake can thoroughly for at least 30 seconds before each use. Hold can completely upside down to dispense. See package insert for dosage information.

Contains:

Sodium Fluoride in a proprietary neutral flavored foam base. 0.9% Fluoride Ion from 2% Sodium Fluoride at a neutral pH. This product does not contain chlorofluorocarbon propellant.

Sweetened with Sucralose. Does not contain Aspartame or Saccharin. Gluten free.

WARNING:

Avoid spraying toward open flame. Contents under pressure. Do not puncture or incinerate. Keep out of reach of children. Store between 55-80° F. Keep from freezing.

EMERGENCY RESPONSE:

Call Infotrac 1-800-535-5053

For professional use only.

Distributed by DMG America LLC
242 South Dean Street
Englewood, NJ 07631
800-662-6383 • www.dmg-america.com